Airport information

I wear a medical device prescribed by my physician.

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Medtronic

Airport security

- The insulin pump **must not** go through the x-ray machine that is used for carry-on or checked luggage.
- The full body scanner is also a form of x-ray. When you go through the body scanner, you need to disconnect the luer set from the insertion site and remove the insulin pump. If continuous glucose monitoring (CGM) is used, remove the sensor and transmitter before the scan.
- To avoid removing the devices, request an alternative screening process that does not use x-ray.
- The insulin pump, infusion set, reservoir, and CGM system can withstand exposure to airport metal detectors used at airport security checkpoints.

In flight

The MiniMed™ insulin pump system can also be used in-flight. When flying in an aircraft, it is important to keep the pump connected to the body and check blood sugar level frequently.

Patient information

Name:

Address:

Country:

Phone number (include country code):

Medical device information

Medical device type:

Device serial number:

© 2023 Medtronic. Medtronic, メドトロニック and Medtronic logo marks are trademarks of Medtronic. Brands marked with ™ are trademarks of a Medtronic company. Third party brands marked with ™* are trademarks of their respective owners.

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In an emergency, please contact

Physician name:
Phone number:
Emergency contact:
Phone number:

I have diabetes

If my behavior is peculiar, if I appear intoxicated, or if I am unconscious, it may be a result of severe low blood sugar.

I am not intoxicated Call for medical assistance

If I am awake and able to swallow, give me a source of sugar (for example: juice, candy, or non-diet soft drink). Do not try to give me food or drink if I am unconscious.

Healthcare professionals

I wear an insulin pump.

- The pump delivers rapid-acting insulin at a constant rate.
- To stop insulin delivery on the insulin pump:
 - Press any button to wake the insulin pump.
 - From the home screen, press ^ to go to the status screen.
 - Press ◎ to **Suspend All Delivery**.
 - Press > to highlight **Yes**.
 - Press ◎ to suspend all insulin delivery.
- Do not** remove the insulin pump battery.
- Do not** remove the insulin pump without medical consent.
- If the insulin pump sounds an alarm, follow the instructions on the insulin pump screen or contact Medtronic Japan 24 hours Support Line for further assistance.