

Medtronic

Rediscover pediatric VV ECMO

You spoke. We listened.

Crescent™ RA
jugular dual lumen catheter

Right atrial placement

Features and benefits

Right atrial design

- Catheter designed to be placed with tip in the right atrium

Flexible technique options

- Includes an introducer for a percutaneous technique or a blunt tip obturator for a surgical cut-down technique


Smaller sizes

- Catheter is available in sizes 13, 15, and 19 Fr

Indications for use

The jugular dual lumen catheter is a single-use dual lumen catheter that provides both venous drainage and reinfusion of blood via the jugular vein. It is indicated for use in adult and pediatric patients with acute respiratory failure requiring veno-venous extracorporeal membrane oxygenation where other available treatment options have failed and continued clinical deterioration is expected, or the risk of death is imminent.

Risks of ECLS include heart, vessel, or lung damage, hypoxia, anemia, infection, hemorrhage, liver or kidney failure, stroke, and death.

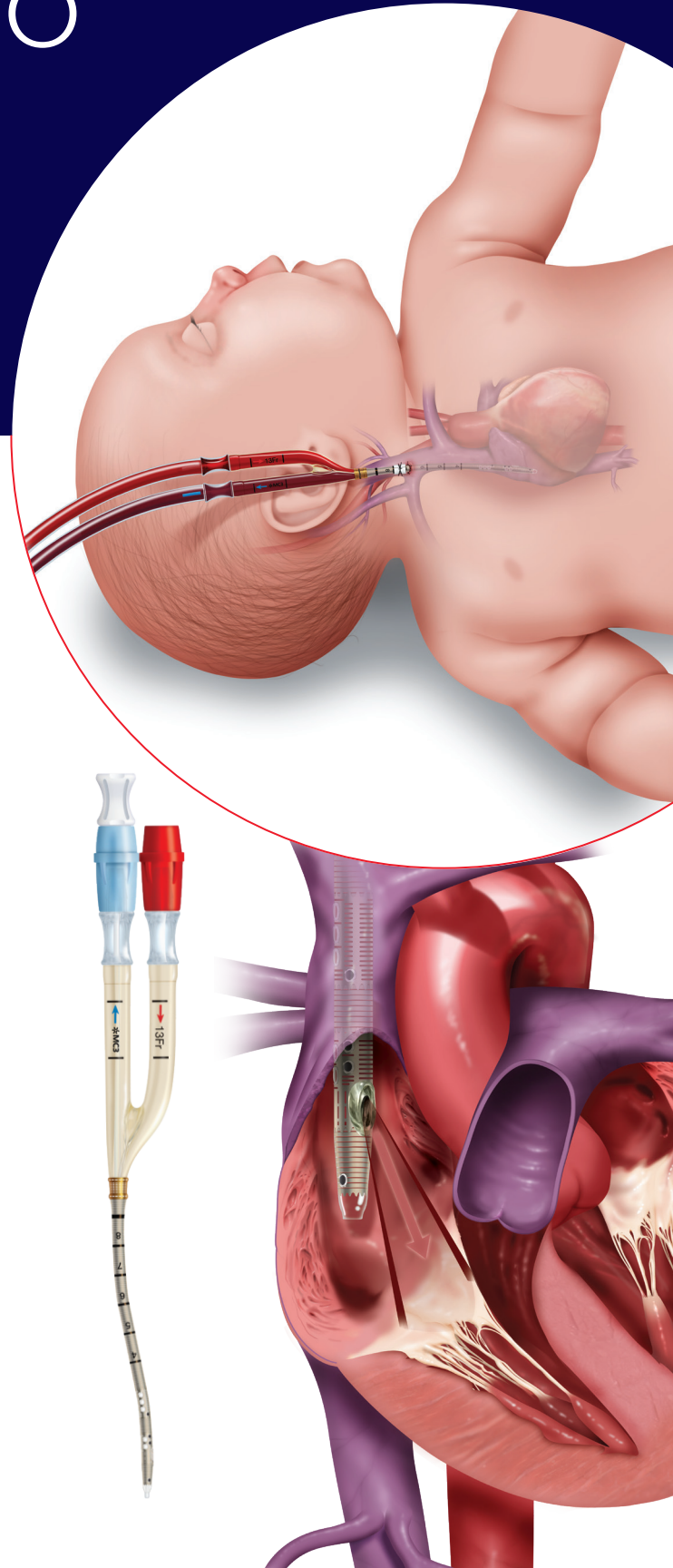

Game-changing flow¹

Correct placement made easier


Radiopaque markers identify drainage sites, reinfusion port, catheter tip, and axial orientation

Optimized flow rates


Crescent-shaped lumen and large reinfusion port help optimize pressure flow performance

Durable securement

Multiple suture points provide additional stability, reducing the chance of twisting or migration when the patient moves

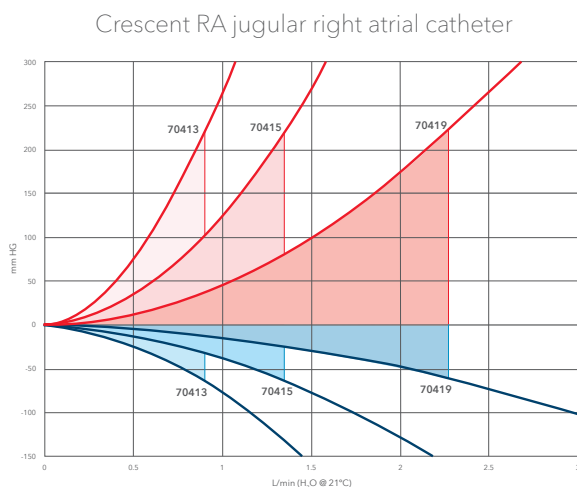

¹ Design verification and validation data on file at MC3. These tests may not be indicative of clinical performance.

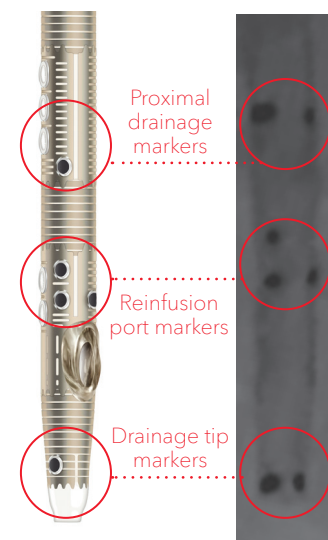


Ordering information

Crescent™ RA jugular right atrial dual lumen catheter

One unit per carton

Code	French size	Insertion length	Connector
70413	13 Fr	8.9 cm	1/4 inch
70415	15 Fr	9.7 cm	1/4 inch
70419	19 Fr	14.5 cm	1/4 inch

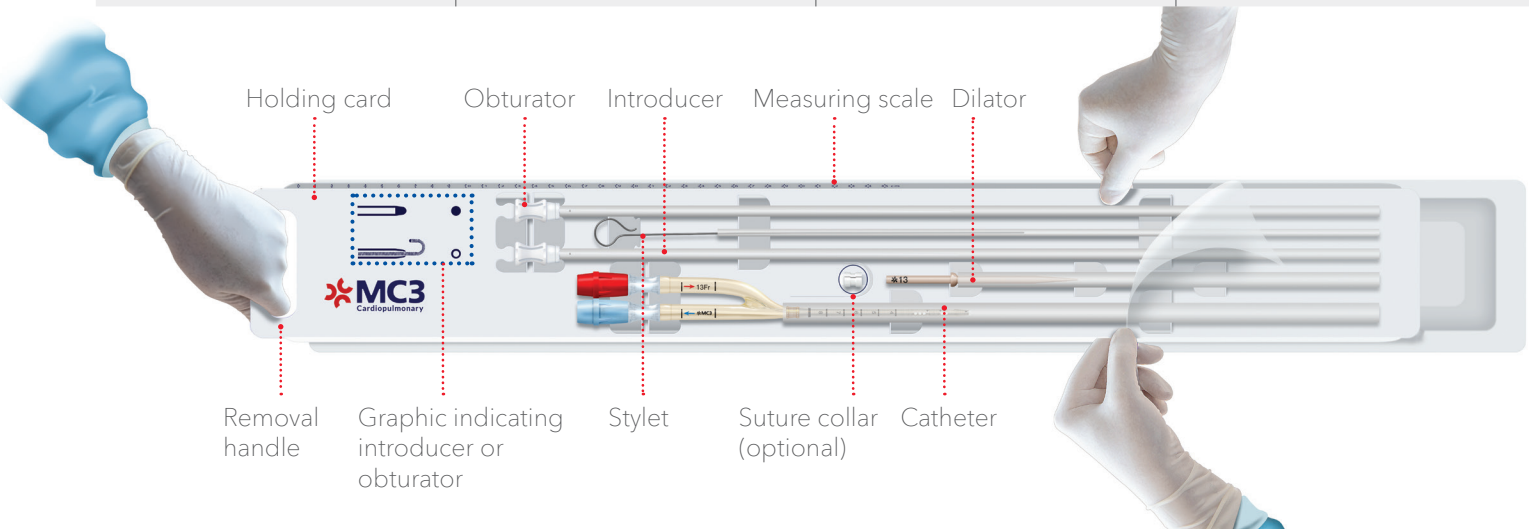

For more information, contact your local Medtronic cannula products representative. U.S. Customer Service: 1-800-328-1357.

Caution: Federal law (USA) restricts this device to sale by or on the order of a physician. For a listing of indications, contraindications, precautions, and warnings, please refer to the Instructions for Use.

Crescent™ RA jugular dual lumen catheter is manufactured by MC3, Inc. and exclusively distributed by Medtronic.

710 Medtronic Parkway
Minneapolis, MN 55432-5604
USA

Toll-free: 800.328.2518
Tel: +1.763.514.4000

medtronic.com

LifeLine
CardioVascular Technical Support
Toll-free: 877.526.7890
Tel: +1.763.526.7890
rs.structuralheart@medtronic.com

©2023 Medtronic. All rights reserved. Medtronic and the Medtronic logo are trademarks of Medtronic. ™*Third-party brands are trademarks of their respective owners. All other brands are trademarks of a Medtronic company.

UC202207936a EN
12/2023

Medtronic