


Medtronic

Chocolate™ PTA
Balloon Catheter

Your PTA
treatment
options just
got sweeter

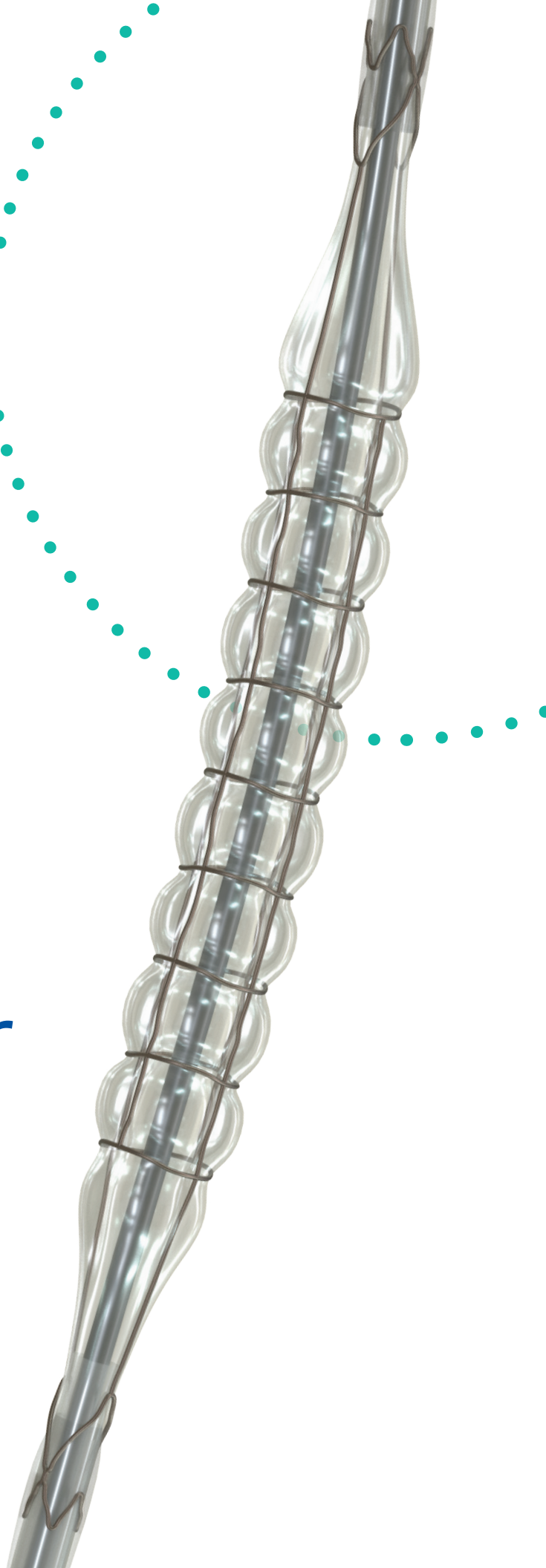

Predictable, uniform, and atraumatic dilatation

Confidently size 1:1 with controlled angioplasty, while reducing the risk of dissections and minimizing the need for bailout stenting with the Chocolate™ PTA balloon.¹ The balloon's unique nitinol constraining structure creates pillows and grooves that provide a predictable, uniform, and atraumatic dilatation.

Braided catheter shaft:

- Designed for pushability to reach and cross lesions.

Nylon, semi-compliant balloon:

- Allows for optimal balloon pillow formation.

Pillows:

- Provides predictable vessel dilatation without cutting or scoring.

Nitinol constraining structure:


- Shields vessel wall from shear (torsional) stress caused by balloon unfolding.
- Enables the even distribution of radial and longitudinal forces during balloon inflation.
- Allows for rapid deflation and uniform rewrap.

Grooves:


- Relieves the stress and stop dissections from propagating.

Tapered tip:

- Enables lower entry profile for optimal lesion access.

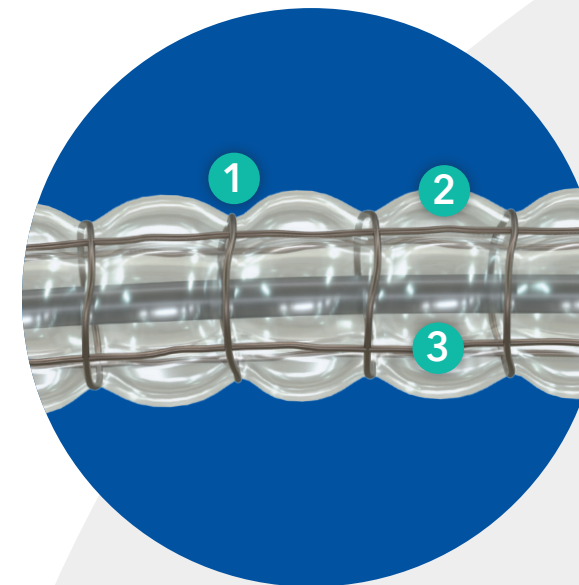

- 1 Pressure relief grooves
- 2 Uniform dilatation pillows
- 3 Nitinol constraining structure

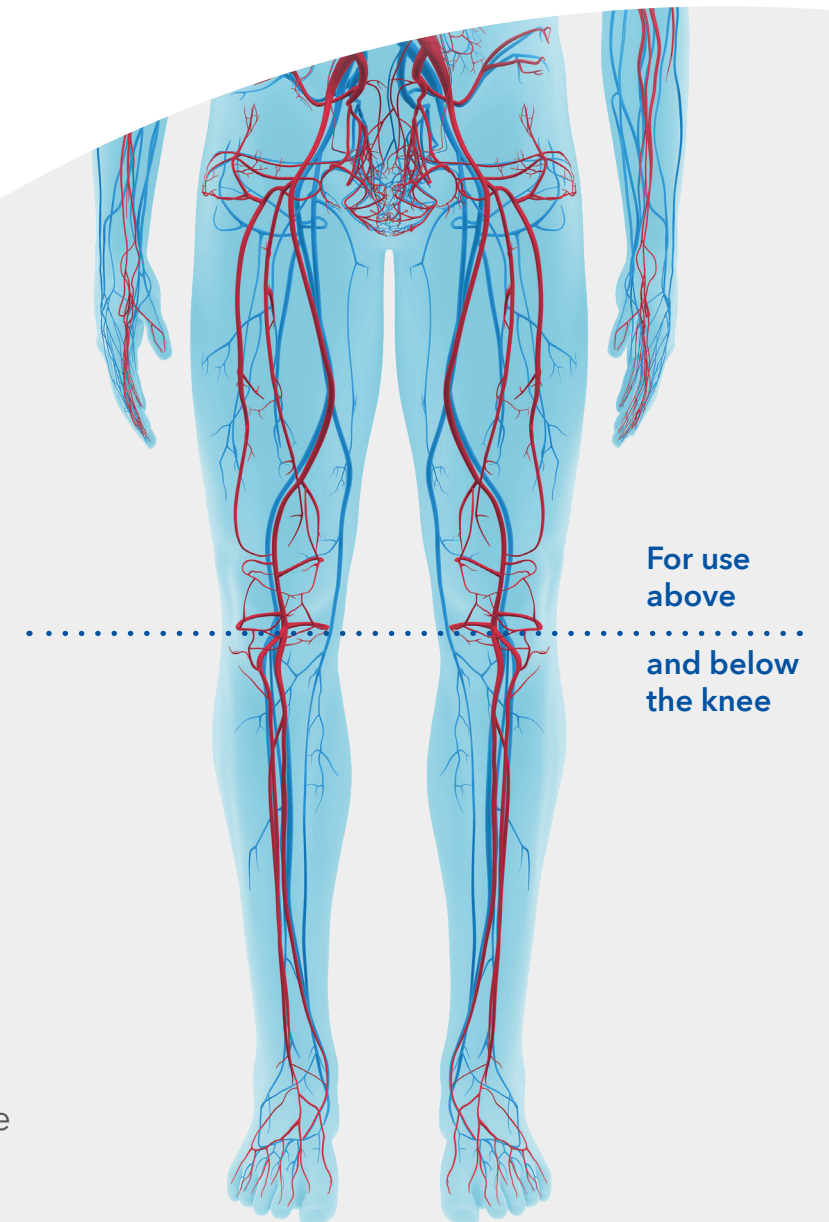

Chocolate™ PTA Balloon

| Ordering information | | | | | | | |
|----------------------|-----------------------|---------------------|----------------------|----------------|-----------------------|------------------------|----------------------------|
| Product code | Balloon diameter (mm) | Balloon length (mm) | Catheter length (cm) | Guidewire (in) | Introducer sheath (F) | Nominal pressure (atm) | Rated burst pressure (atm) |
| CB1415025040 OTW | 2.5 | 40 | 150 | 0.014" | 5 | 9 | 14 |
| CB1415025080 OTW | | 80 | | | | | |
| CB1415025120 OTW | | 120 | | | | | |
| CB1415030040 OTW | 3.0 | 40 | | | | | |
| CB1415030080 OTW | | 80 | | | | | |
| CB1415030120 OTW | | 120 | | | | | |
| CB1413535040 OTW | 3.5 | 40 | 135 | 0.014" | 5 | 9 | 14 |
| CB1413535080 OTW | | 80 | | | | | |
| CB1413535120 OTW | | 120 | | | | | |
| CB1413540040 OTW | 4.0 | 40 | | | | | |
| CB1413540080 OTW | | 80 | | | | | |
| CB1413540120 OTW | | 120 | | | | | |
| CB1812050040 OTW | 5.0 | 40 | 120 | 0.018" | 6 | 6 | 12 |
| CB1812050080 OTW | | 80 | | | | | |
| CB1812050120 OTW | | 120 | | | | | |
| CB1812060040 OTW | 6.0 | 40 | | | | | |
| CB1812060080 OTW | | 80 | | | | | |
| CB1812060120 OTW | | 120 | | | | | |

¹ Medtronic data on file, Final Study Report The Chocolate BAR CLR782.

Risks may include: abrupt or subacute closure, pseudo-aneurysm, thrombosis, allergic reaction to device materials or procedural medications, arterial injury (such as dissection, perforation, or rupture), embolism, hematoma, and hemorrhage.

Important Information: Indications, contraindications, warnings, and instructions for use can be found in the product labeling supplied with each device.

Indications for Use: The Chocolate™ PTA Balloon Catheter is intended for balloon dilatation of lesions in the peripheral vasculature, including the iliac, femoral, iliofemoral, popliteal, infra-popliteal, and renal arteries.

CAUTION: Federal (USA) law restricts these products for sale by or on the order of a physician.

Medtronic

[medtronic.com/chocolate](https://www.medtronic.com/chocolate)

UC201709657b EN ©2023 Medtronic. All rights reserved. Medtronic, Medtronic logo, and Engineering the extraordinary are trademarks of Medtronic.™™Third-party brands are trademarks of their respective owners. All other brands are trademarks of a Medtronic company. 03/2023