

Set up guide

Step 1 Choose a blade



TruClear™ Soft Tissue Shaver Mini

Polyp, RPOC, Visual D&C

TruClear™ control unit
should read <1500>

TruClear™ Dense Tissue Shaver Mini

Fibroids

TruClear™ control unit
should read 2500

Step 2 Check the handpiece

Make sure the Thumb Lever/Suction
Lever is in the OFF position

Step 3 Insert shaver


Position the blade so that the cutting
window is visible on screen

Step 4 Window lock

Hold the “Window Lock”
button down (upper
right hand button on foot
pedal) until inner black
line is inside the outer
black line on the shaver



TruClear™
Soft Tissue
Shaver Mini

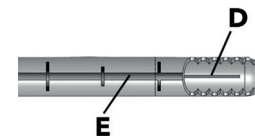

Press the “Window Lock” button on the
TruClear™ footswitch to align the axial
reference lines (D and E)
on the inner and outer
tubes respectively†

Watch video here for
more information



Step 5 Turn on suction

On the handpiece, turn the thumb
lever/suction lever to the ON position

Step 6 Begin resection

Place cutting window against the
pathology and depress EITHER foot pedal

†Always refer to the Instructions For Use included with the product for complete indications, instructions, warnings, precautions, and contraindications.

[medtronic.com/truclear](https://www.medtronic.com/truclear)