

# STEP-BY-STEP GUIDE

## ReliaTack™ Articulating Reloadable Fixation Device




## LOADING

Before loading, the ReliaTack™ device must be in reload mode, with the red toggle button exposed.



1 Ensure that the device is in the straight, unarticulated position.

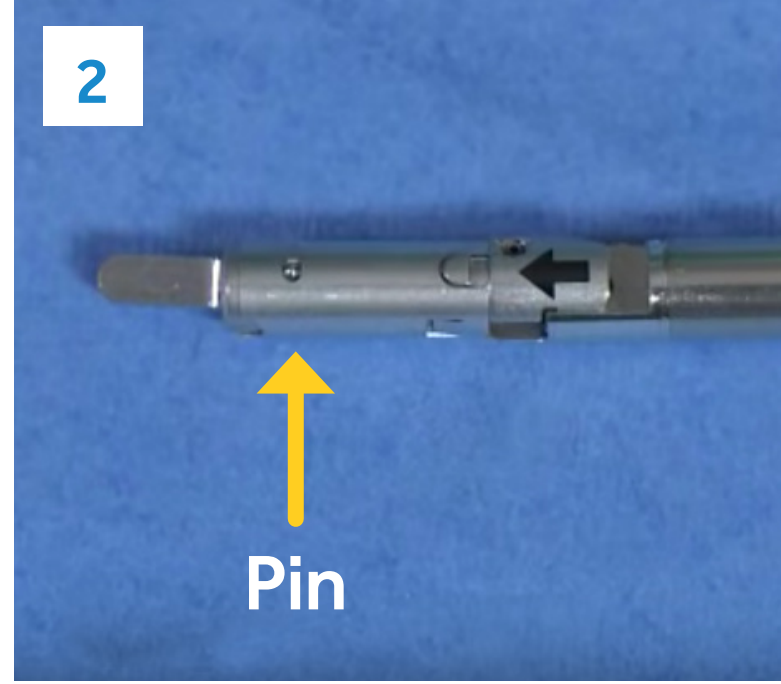


2 Pin must be in the vertical position. If not vertical, push the red fire button, cycle the device until the pin is vertical, and push the green load button once vertical.




3 Remove the reload from the blister by grabbing the distal end of the reload and lifting up. Important: Do not grab the shipping wedge.




4 Pull and hold the blue lock switch button in the back position.

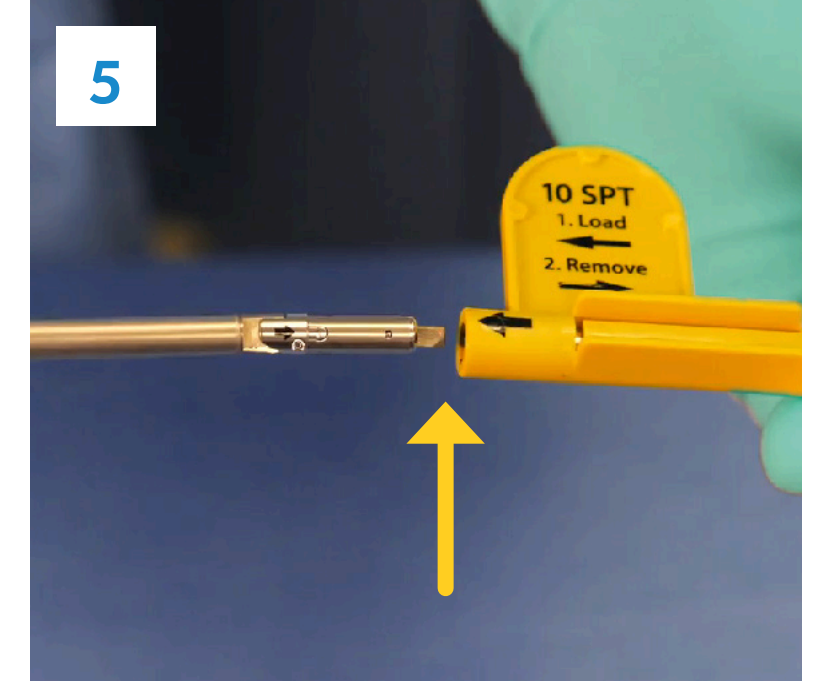


5 Hold the blue lock switch button and insert the pin located at the distal end of the shaft into the reload.

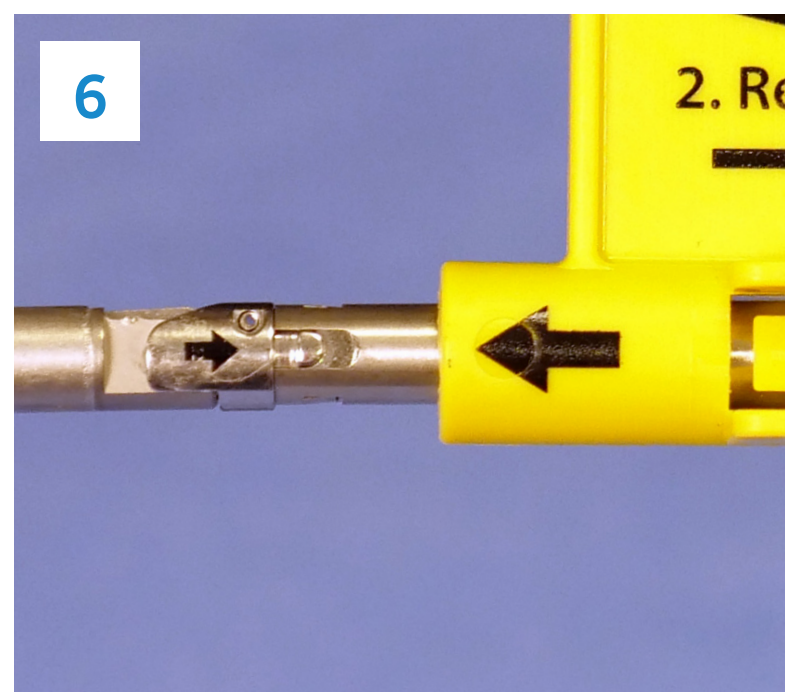

6 Ensure that the arrow on the shaft aligns with the shipping wedge arrow. Push the reload onto the distal end of the shaft until the two arrows meet.

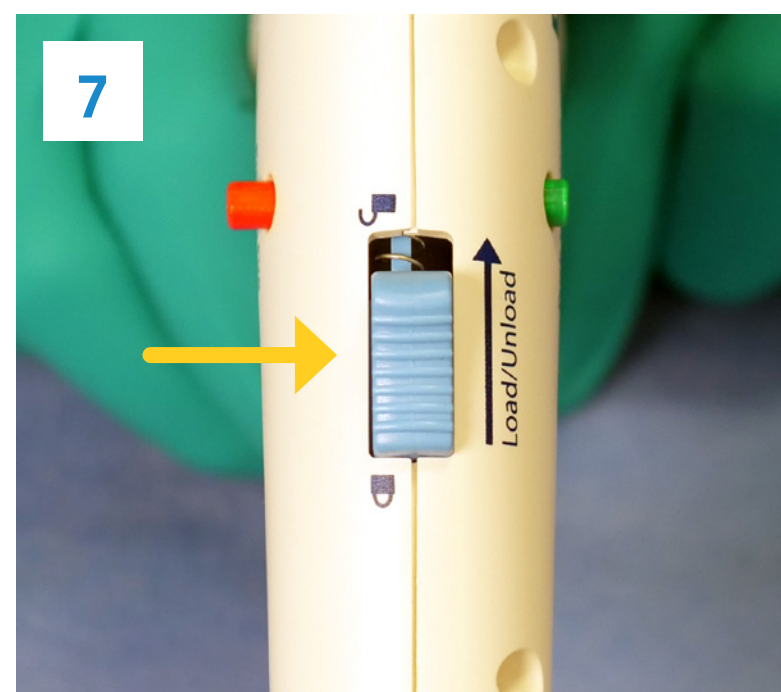


7 Release the blue lock switch button, allowing it to return to the forward position.

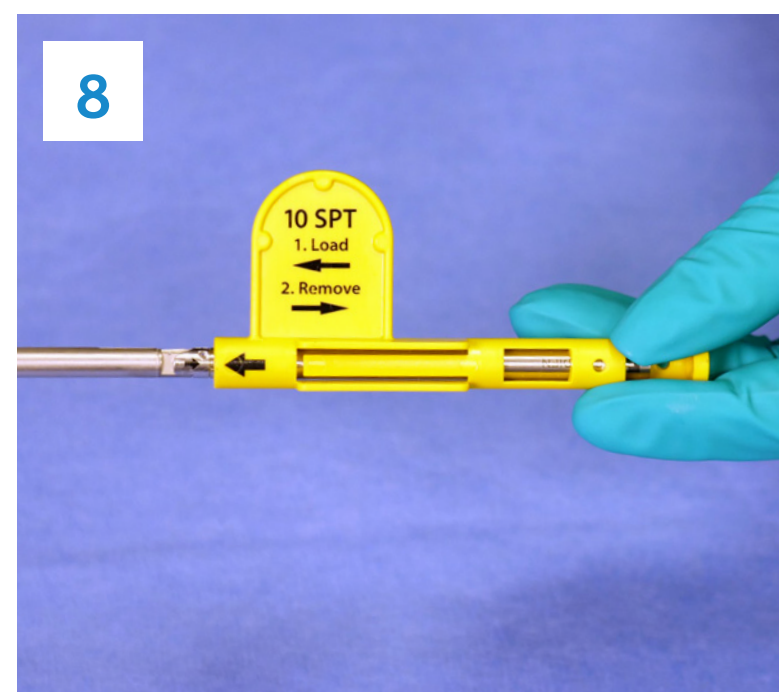

8 Gently pull on the reload to confirm that the device is loaded.




9 Remove the shipping wedge once the reload is properly locked and loaded.

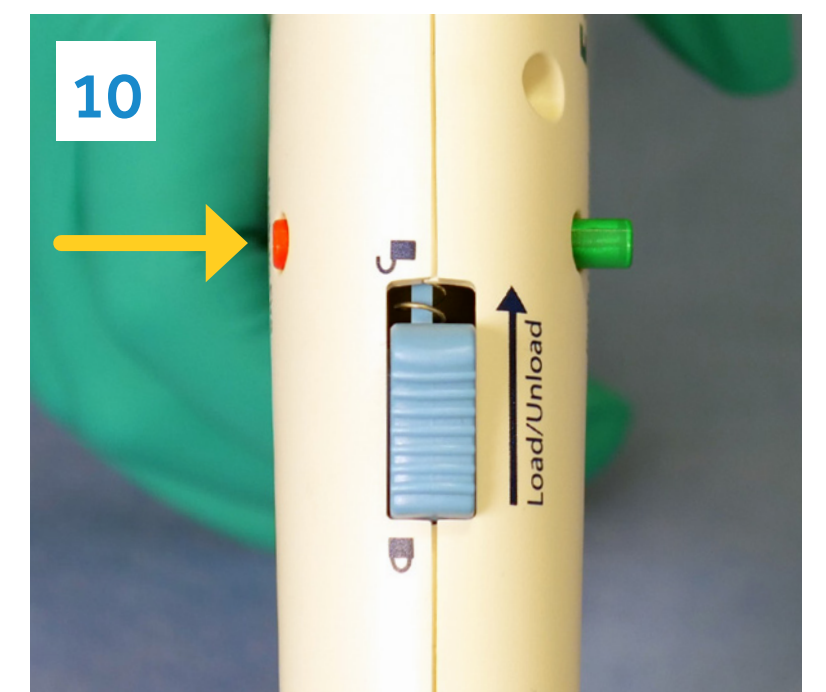


10 Push the red fire button and hand the device to the surgeon.


## UNLOADING

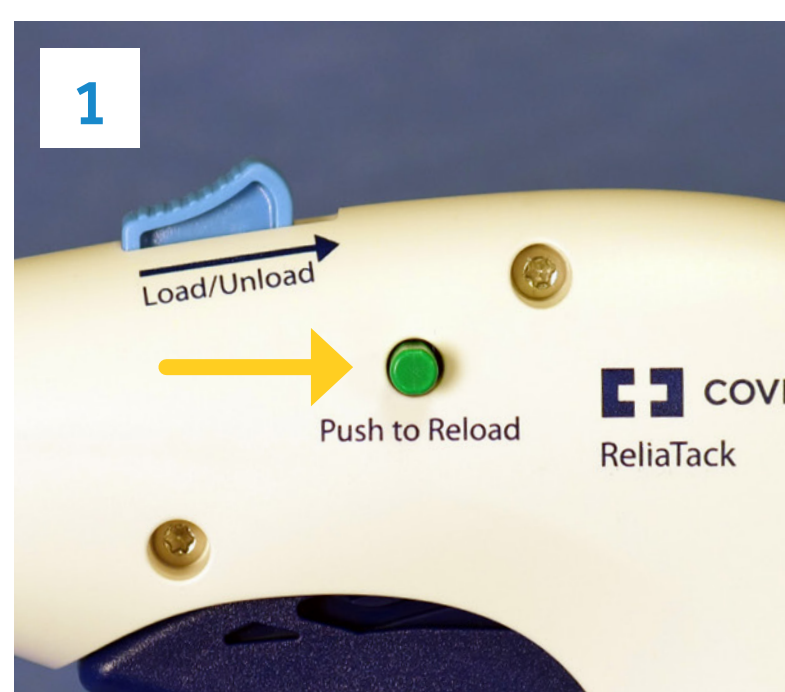

1 Push the green load button to expose the red portion of the toggle button and ensure the device is in the straight position.

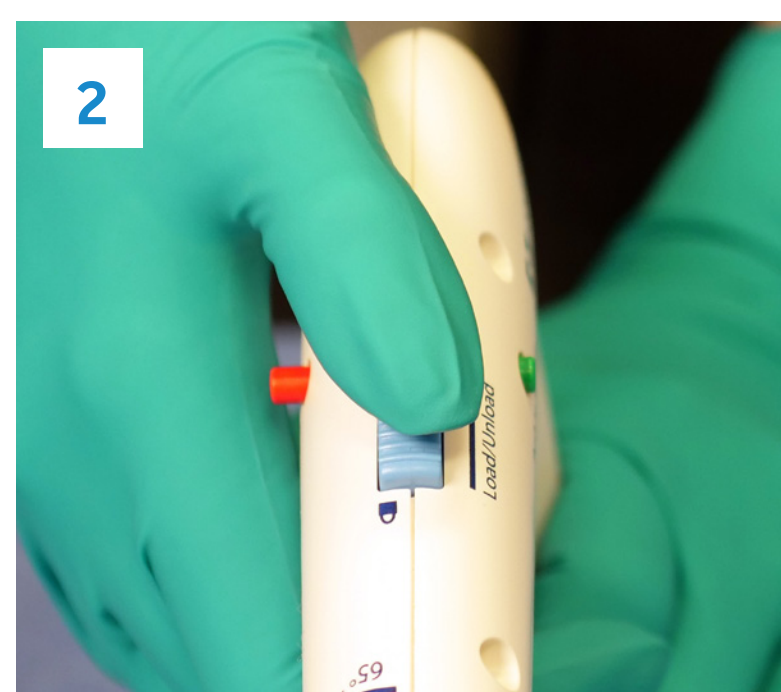

2 Pull and hold the blue lock switch button in the back position.

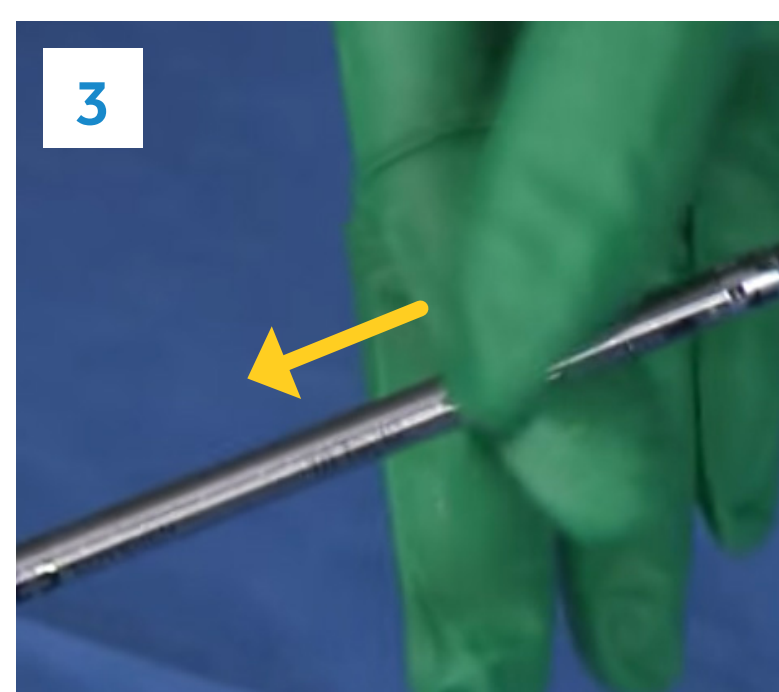

3 Pull the reload off of the shaft of the device and discard.  
**Warning: Do not reuse.**

## HERNIA CARE

**Mesh. Fixation. Permacol™. Dissection.**

Our comprehensive product portfolio can enhance your hernia repair procedures.

Always refer to the instructions for use included with the product for complete indications, contraindication, warnings, and precautions.

555 Long Wharf Drive  
New Haven, CT 06511

800.722.8772  
508.261.8000

[medtronic.com/reliatack](http://medtronic.com/reliatack)

© 2021 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic. ™\* Third party brands are trademarks of their respective owners. All other brands are trademarks of a Medtronic company. 02/21 - US-HR-2100017

**Medtronic**  
Further, Together



# STEP-BY-STEP GUIDE

## ReliaTack™ Articulating Reloadable Fixation Device




### LOADING

Before loading, the ReliaTack™ device must be in reload mode, with the red toggle button exposed.



1 Ensure that the device is in the straight, unarticulated position.

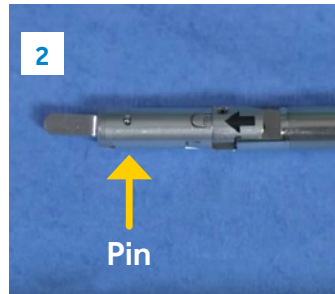


2 Pin must be in the vertical position. If not vertical, push the red fire button, cycle the device until the pin is vertical, and push the green load button once vertical.




3 Remove the reload from the blister by grabbing the distal end of the reload and lifting up. Important: Do not grab the shipping wedge.




4 Pull and hold the blue lock switch button in the back position.

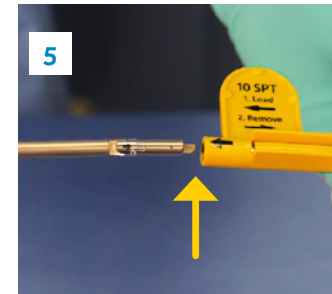

5 Hold the blue lock switch button and insert the pin located at the distal end of the shaft into the reload.

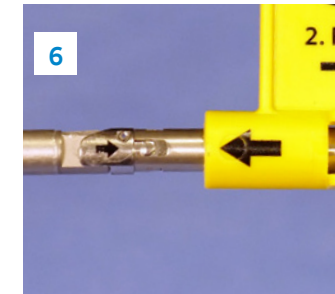

6 Ensure that the arrow on the shaft aligns with the shipping wedge arrow. Push the reload onto the distal end of the shaft until the two arrows meet.

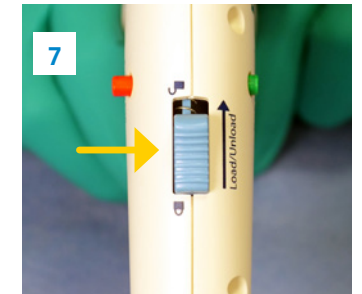


7 Release the blue lock switch button, allowing it to return to the forward position.




8 Gently pull on the reload to confirm that the device is loaded.



9 Remove the shipping wedge once the reload is properly locked and loaded.

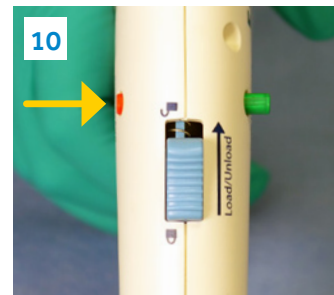


10 Push the red fire button and hand the device to the surgeon.

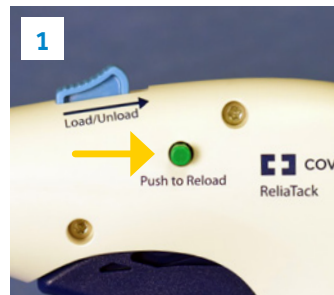

11 Push the green load button to expose the red portion of the toggle button and ensure the device is in the straight position.



12 Pull and hold the blue lock switch button in the back position.

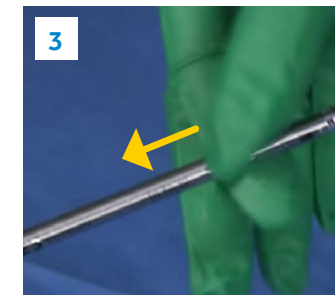

13 Pull the reload off of the shaft of the device and discard.

**Warning: Do not reuse.**

### UNLOADING

**HERNIA CARE**  
Mesh. Fixation.  
Permacol™.  
Dissection.

Our comprehensive product portfolio can enhance your hernia repair procedures.

Always refer to the instructions for use included with the product for complete indications, contraindication, warnings, and precautions.

555 Long Wharf Drive  
New Haven, CT 06511  
800.722.8772 508.261.8000

[medtronic.com/reliatack](http://medtronic.com/reliatack)

© 2021 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic. ™\* Third party brands are trademarks of their respective owners. All other brands are trademarks of a Medtronic company. 02/21 - US-HR-2100017

**Medtronic**  
Further, Together