## Supplement Paper RespArray

EN	Supplement, page 1
DE	Beiblatt, Seite 9
IT	Supplemento, pagina 18
FR	Addendum, page 27
ES	Suplemento, página 36
PT-BR	Suplemento, página 45
NL	Supplement, pagina 54

## Supplement to the User Manual

---

Medtronic P/N: RespArray01AHA, RespArray01IEC

This supplement is used for RespArray.

- This supplement must be kept with the user manual of the medical device.
  - The following information supplements the relevant chapters in the user manual.
- 

## Chapter 3 Basic Operation

---

### 3.2 Operating and Navigating

	Battery status - more symbols for different situations should refer to section 22.3
	Wireless network is connected and the signal quality is good.
	Wireless network is not connected or the signal quality is poor.
	Wired network is connected
	Wired network is not connected

#### 3.2.1.2 Shortcut Keys

Clear the NIBP physiological alarm indications

## Chapter 4 Networked Monitoring

---

### 4.2.3 Setting the Wireless Network

The monitor can optionally be equipped with the Wi-Fi module. To configure the settings on the monitor following the steps below before connecting the monitor to a wireless network:

1. Select **Menu > System > User Maintain**, and input the password.
2. Select **Network Setup > Connections**. Set **Connection Type** to **WLAN**.
3. Select **WLAN** and set **Frequency Band** to **2.4GHz**, **5GHz** or **Auto**. Restart the monitor and the available networks will be listed in **WLAN** window. Choose a network from the window, in which the user can check the network's encryption information. The user will be prompted to enter the password of that network if a password is required. After entering the password, click **Connect**.
4. Or, select **Other** and input the information as needed. Click **Connect**.

- ◆ For WPA2-PSK, input the network name and password.
  - ◆ For WPA2-EAP, input the network name, EAP method (TLS/TTLS/PEAP), inner authentication (EAP-MSCHAPV2/any), domain, username and password as required. When the EAP method is set to TLS, import the certificate according to Section *Managing Certificates* and then select the corresponding certificate from the **Certificate List**.
5. Select **Config** to set **Mode**, **Local IP address**, **Subnet Mask** and **Gateway**.

If the monitor is successfully connected to the selected network, it will be indicated by the icon  in the system information area, and the Wi-Fi currently in use will be displayed at the top of the **WLAN** window. The meanings of the networking state symbols are explained below:

	Wi-Fi signal intensity: Level 4
	Wi-Fi signal intensity: Level 3
	Wi-Fi signal intensity: Level 2
	Wi-Fi signal intensity: Level 1

Click  to review the historically connected network. After choosing certain network, the user can select **Delete** or **Join**.

If the encryption information of the currently connected network is modified, the network will automatically disconnect and attempt to reconnect. At this time, click  first to ignore this network and then connect manually. For an unconnected network, if the encryption information or SSID is modified and the user attempts to connect it, the user needs to disconnect the currently connected network and click  to select the updated network.

Select **Disconnection Statistics** to view details of the network disconnection and click **Export** to export the statistics data to the specified USB drive.

In **Wi-Fi Maintain** menu, enable the **Background Scan Switch** and select **Background Scan Channel** to search for better WLAN in specified channels. Roaming threshold and lag can also be viewed.

The following symbols may appear when configuring Wi-Fi:

Symbol	Description
	View historically connected network
	Refresh network list
	The network is password protected.

**NOTE**

The obstacles such as the presence of the main walls and their number may interfere with data transmission and even cause data loss.

#### NOTE

When signal intensity is level 2 or less, signal may be unstable and quality of the signal transmission may be degraded.

#### NOTE

When the monitor is connected to Gateway via the wireless network, the user should set the router to a secure encryption/authentication and use a high complexity, non-dictionary password.

#### NOTE

It is not recommended to use Wi-Fi and Bluetooth simultaneously; otherwise, co-channel interference may be arisen.

#### WARNING

Before monitoring patient, Connection Type (Wired or WLAN) should be selected and is not allowed to switch during monitoring. Otherwise, the data may not be transmitted to the central monitoring system.

#### WARNING

If Wi-Fi is unavailable, restart WLAN to restore Wi-Fi function under the precondition of ensuring the patient safety.

#### WARNING

If enterprise encryption uses an inappropriate self-signed certificate, there is a risk of information leakage! Do not connect the monitor to untrusted hotspots!

## Chapter 5 Alarms

---

### 5.3.1 Changing Technical Alarm Priority Settings

To set the priority of the SpO<sub>2</sub> Sensor Off alarm and SpO<sub>2</sub> No Sensor alarm, select **Menu > System > User Maintain** and input user maintenance password > **Parameter Maintain > SpO<sub>2</sub> > SpO<sub>2</sub> Sensor Off Alarm Level** or **SpO<sub>2</sub> No Sensor Alarm Level**.

## Chapter 8 Managing Profiles

---

### 8.4 Transferring Profile

When installing several monitors with identical user profiles, it is not necessary to set each

monitor separately. Use a USB flash disk to transfer the profile from monitor to monitor.

To export the current monitor's profile, follow these steps:

1. Connect the USB flash disk to the monitor's USB port.
2. Select **Menu > Profile > Transfer Profile** and input the required password.
3. Choose the removable device and click **Export** to enter the preview interface. Select a profile from the profile list and click **Confirm**.

To import the configuration from the USB flash disk to the monitor, follow these steps:

1. Connect the USB flash disk to the monitor's USB port.
2. Select **Menu > Profile > Transfer Profile** and input the required password.
3. Choose the removable device and click **Import** to enter the preview interface. Select the profile from the profile list and click **Confirm**.

You may need to transfer the network configuration during the installation. Therefore, the monitor allows you to transfer and apply a desired configuration to ensure that batch of monitors share the identical network configurations.

To select and apply the network configuration, follow these steps:

1. Select **Menu > Profile > Transfer Profile** and input the required password.
2. Choose **Select Network Configuration** and select desired configuration.
3. Select **Apply Network Configuration** to apply the current settings.

---

## Chapter 16 Monitoring Microstream™ Capnography

### 16.8 CO<sub>2</sub> Functional Test

If the CO<sub>2</sub> module's calibration or maintenance is overdue, the monitor displays related information on the Turn-on Screen and **Parameter Maintain** of **User Maintain** screen.

The **Parameter Maintain > CO<sub>2</sub>** screen displays the **Remaining Hours to Next Calibration** and indicates a **CO<sub>2</sub> Calibration Required** condition to the user. On the Turn-on Screen, a **CO<sub>2</sub> Calibration Required** message is shown if CO<sub>2</sub> calibration or maintenance is overdue.

To calibrate the CO<sub>2</sub> module, or to check its calibration, use the **Calibrate** and **Calibration Check** on the **Parameter Maintain > CO<sub>2</sub>** Screen.

The monitor prompts the user through various steps of calibration. The CO<sub>2</sub> Cal screen gives user two options: **Calibrate** or **Calibration Check**. It also gives the user information on the **Last Calibration Date**, **Last Calibration Time** and **Calibration should be performed by: XXXX-XX-XX**. Performing Calibration results in CO<sub>2</sub> module finetuning internal parameters to compensate for any component drifts, while Calibration Check reports to the user whether the current parameters still produce accurate CO<sub>2</sub> readings.

### 16.9 CO<sub>2</sub> Calibration Requirements

The CO<sub>2</sub> unit is calibrated when it leaves the factory. It is recommended that you calibrate the monitor within two weeks of the **CO<sub>2</sub> Calibration Required** message appearing on the monitor.

#### **PRECAUTION**

**The calibration must be performed with a manufacturer authorized Calibration Kit containing a gas mixture of 5% (+/- 0.05%) CO<sub>2</sub> 21% (+/- 1%) O<sub>2</sub> and Bal N<sub>2</sub> and the authorized connecting accessories (“T” piece).**

#### **NOTE**

**Dispose of calibration gases according to local regulations for disposal of gases.**

For the US region, a manufacturer approved ETCO<sub>2</sub> Calibration Kit can be purchased directly from Air Liquide (Scott Medical) using manufacturer part number Z03N1748BDC002. To purchase the Calibration Kit (twin package) contact Air Liquide Customer Service at 800-233-4334, option 1, then option 2 or email at [customerservice.healthcare@airliquide.com](mailto:customerservice.healthcare@airliquide.com).

The Calibration Kit (twin package) includes:

- 2 Calibration Gas containing 5% CO<sub>2</sub>, 21% O<sub>2</sub> Bal N<sub>2</sub>
- 2 Tubing Adapter (“T” Piece)
- 2 Calibration Line (Calibration Sampling Line)

For the EU and other regions, contact Medtronic Technical Support or a local Medtronic representative to seek alternative calibration gas sources.

### **16.9.1 CO<sub>2</sub> Calibration Check**

Depending on institutional policy and procedure, the CO<sub>2</sub> **Calibration Check** function within the RespArray monitor can be leveraged to determine if CO<sub>2</sub> measurement is within the accepted limits. However, this process does not calibrate the device.

The calibration check must be performed with a manufacturer authorized Calibration Kit containing a gas mixture of 5% CO<sub>2</sub>, 21% O<sub>2</sub> and Bal N<sub>2</sub> and the authorized connecting accessories (“T” piece).

If this process is performed while a battery powers the monitor, make sure that the battery is fully charged.

#### **NOTE**

**At any stage in the Calibration Check procedure, you can go back to the first screen by clicking the BACK button.**

Prior to checking the calibration, verify that the Calibration Line supplied with the Calibration Kit is firmly attached.

The **Calibration Check** process includes the following steps:

1. Click the **Menu** shortcut.
2. Select **System > User Maintain**, input the required password.
3. Click **Calibration Check** button in **Parameter Maintain > CO<sub>2</sub>** screen.
4. The instructions on the screen will guide you regarding the rest of the process, including results.

#### **NOTE**

If the option **Calibrate Check** is not selectable or grayed out, the **CO<sub>2</sub>** parameter is in **Standby**. The **CO<sub>2</sub>** parameter must be set to **Measure** to allow for calibration check. Refer to the following menu structure to change the **CO<sub>2</sub>** parameter status to **Measure**: **Parameter > Param Switch > Param Setup > CO<sub>2</sub> > Work Mode > Measure**.

### **16.9.2 CO<sub>2</sub> Calibration**

Calibration shall be performed as follows:

1. Click the **Menu** shortcut.
2. Select **System > User Maintain**, input the required password.
3. Select **Parameter Maintain**, then, select **CO<sub>2</sub>** parameter.
4. The **Parameter Maintain** screen displays the Remaining Hours to Next Calibration. It also provides the user with information on the Last Calibration Date, Last Calibration Time, and when Calibration should be performed by.
5. The instructions on the screen will guide you regarding the rest of the process, including results, until you reach the screen which will provide you with calibration results.

#### **NOTE**

If the option **Calibrate** is not selectable or grayed out, the **CO<sub>2</sub>** parameter is in **Standby**. The **CO<sub>2</sub>** parameter must be set to **Measure** to allow for calibration. Refer to the following menu structure to change the **CO<sub>2</sub>** parameter status to **Measure**: **Parameter > Param Switch > Param Setup > CO<sub>2</sub> > Work Mode > Measure**.

## **Chapter 21 Other Functions**

---

### **21.1 Nurse Call**

The monitor provides a dedicated nurse call port to output a nurse call signal when a user-defined alarm occurs. To establish communication between the nurse call system and the monitor, use the dedicated nurse call cable to connect the hospital's nurse call system to the monitor's nurse call port.

Alarms are sent to the nurse call system given the following conditions are met:

- The nurse call system is enabled.
- A user-defined alarm occurs.
- Alarms are not paused or reset.

The nurse call function can be set through the following steps:

1. Select **Menu > System > User Maintain**, input the password.
2. Select **Other Setups** and enable **Nurse Call**.
3. Select **Signal Type**:
  - ◆ **Continuous**: the nurse call signal lasts until the alarm ends, which means the duration of a nurse call signal is equal to that of the alarm condition.
  - ◆ **Pulse**: the nurse call signal is a pulse signal, and the duration of each pulse is user-defined. When multiple alarms occur simultaneously, only one pulse signal is produced. If an alarm occurs but the previous one is not cleared, a new pulse signal will also be produced. Set **Pulse Width** to 1 s, 2 s, or 3 s, as desired.
4. Select **Contact Type** to set the work mode of the nurse call relay.
5. Select **Alarm Level** to set the priority of alarms sent to the nurse call system.
6. Select **Alarm Type** to set the type of alarms sent to the nurse call system.

**NOTE**

Before using nurse call function, please ensure its proper operation.

**NOTE**

If you need to connect the monitor to the nurse call system, a dedicated nurse call cable is required. For more information, contact the service personnel for assistance.

**NOTE**

Do not rely exclusively on the nurse call system for alarm notifications. Remember that the most reliable alarm notification combines audible and visual alarm indications with the patient's clinical condition.

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Shanghai International Holding Corp. GmbH
Distributed by Covidien LLC,