


SonarMed™ system checklist and alarm suggestions

Shift-change checklist:

- 1 Plugged in and powered on
- 2 Sensor and ETT size match
- 3 ETT length is set correctly; adjust if cut. Green ball is in center of red box in waveform screen.
- 4 O₂ range matches ventilator FiO₂

ETT ID size	Estimated tracheal length	Average ETT tip position above carina	ETT low alarm	ETT high alarm
2.5	2.2-4.0 cm	1.0-1.5 cm	0.5 cm	1.0 cm
3.0	4.0-4.7 cm	1.0-2.0 cm	1.0 cm	1.2 cm
3.5	4.7-5.5 cm	1.0-2.5 cm	1.5 cm	1.5 cm

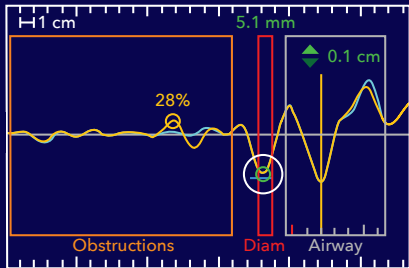

10

