

Medtronic

Appose™ ULC skin stapler

Single-use skin stapler with stainless steel staples

Features

- Two staple width/height options
- Clear plastic cartridge for enhanced staple count visibility

Benefits


- Wound closure flexibility
- Enhanced cosmesis

Proper positioning

The Appose™ ULC skin stapler is designed to be fired from multiple angles. It has an ejector spring that automatically disengages the staple to facilitate instrument removal in any direction.




Appose™ ULC skin stapler

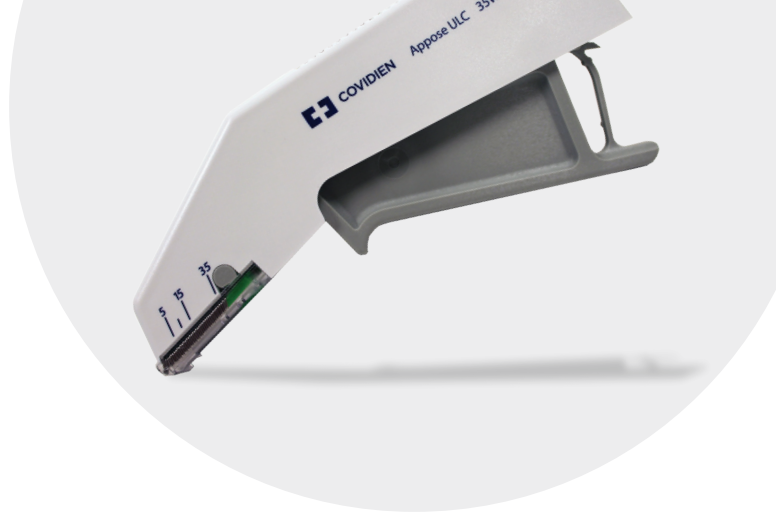

Staple formation

The flat, dimpled staple top span is designed to ensure proper staple formation. To use the skin stapler properly, squeeze the trigger completely and release.

Do not pull or lift up the stapler when releasing the trigger. Failure to completely squeeze the handle can result in incomplete staple formation and insufficient wound approximation.

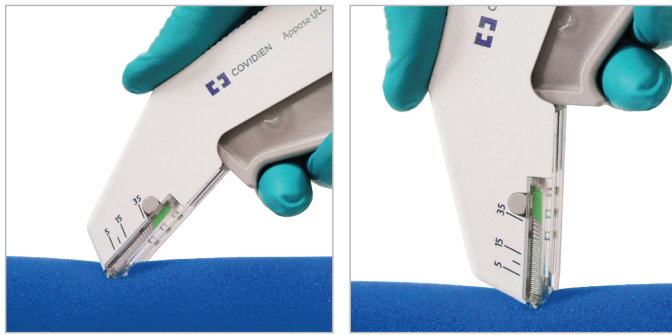

Fully squeeze and release trigger **before** lifting.



Do not lift or rotate until staple is released.

Staple specifications

Appose™ ULC single-use skin staplers are available in two different staple sizes: regular and wide. In the closed position, the regular staple has a width of 5.88 mm and leg length of 4.10 mm; the wide staple has a width of 6.50 mm and leg length of 4.10 mm.

	Size	Closed position	Wire
35R	10.87 mm 2.9 mm	5.88 mm 4.1 mm	0.57 mm
35W	13 mm 3.25 mm	6.5 mm 4.1 mm	0.56 mm

Ordering information

Order code	Description	Units per box
8886803512	Appose™ ULC skin stapler with 35 regular staples	12
8886803712	Appose™ ULC skin stapler with 35 wide staples	12
150462	Premium™ single-use skin staple remover	12

Stainless steel composition of staple

Carbon	0.03%
Chrome	16.0 -19.0%
Nickel	10.0 -15.5%
Manganese	2.0%
Silicon	1.0%
Sulphur	0.03%
Phosphorus	0.045%
Molybdenum	2.0 - 3.0%
Nitrogen	0.10%
Copper	0.50%
Iron	Balance

Important: Please refer to the package insert for complete instructions, contraindications, warnings, and precautions.

©2024 Medtronic. Medtronic, Medtronic logo, and Engineering the extraordinary are trademarks of Medtronic. All other brands are trademarks of a Medtronic company. 04/2024 - US-ST-2000085 - [WF#1124251]